

N.B.B. ORTHOTICS, INC.

3115 Long Beach Road • 3rd Floor
Oceanside, New York 11572
(516) 594-9066
Fax (516) 594-9067

NEIL B. BERGMAN, C.O.
CERTIFIED ORTHOTIST

American Board Certified
Member of A.A.O.P.

**CUSTOM LONG LEG BRACE
PROPER APPLICATION**

- * APPLY STOCKINETTE TO ENTIRE LEG FROM TOES TO GROIN LEVEL.
- * LIFT LEG AND POSITION OPEN BRACE UNDER LEG SO THAT PATIENTS HEEL IS TOUCHING FOOT PLATE OF BRACE.
- * LOWER LEG INTO BRACE.
- * POSITION FRONT THIGH AND SHELL INTO PLACE.
- * CLOSE STRAPS FROM BOTTOM UP.
- * SLIDE LEATHER BELT UNDER PATIENTS WAIST AND CLOSE OVER STOMACH.
- * APPLY SHOE OVER FOOT AND BRACE.
- * BRACE CAN BE UNLOCKED AT THE KNEE BY SIMPLY LIFTING UP THE DROP LOCK.
- * BRACE CAN BE LOCKED AT KNEE BY SIMPLY LOWERING BOTH DROP LOCKS.